# ADD™ anterior distraction device

## ADDplus™ anterior distraction device plus

CS 2256-xx

Supplement für OP-Technik  
Supplement for Surgical Technique

### Distraktionsinstrument Distractor

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The complete distractor  
(CS 2256) consists of:

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First the head (CS 2269-12,  
-14 or -16) is mounted on  
the implant. Both connected  
components are then mount-  
ed on the distractor.

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**Important:** Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to: UH 1100 „Processing manual implants and instruments“. For individual instruments which are correspondingly marked (⚠), we refer to the „Assembly and disassembly instructions with special cleaning instructions“. These can be downloaded any time free of charge at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)

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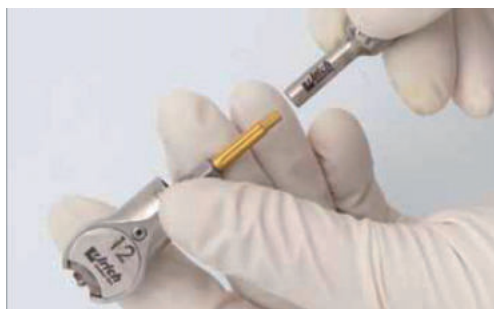
#### Mounting the head in the implant

Select the appropriate head size for the selected implant (e.g. CS 2269-12 for an implant with a diameter of 12 mm). Mount the assembly assistant (CS 2256-02) on the posterior working end of the head (CS 2269-xx) (see figure).

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The connection of the anterior working end of the head (CS 2269-xx) is established in the following way on the implant (CS 22xx): for fixation, the screw end protruding from the front of the head (CS 2269-xx) engages in one of the holes on the implant provided for the locking screw. The pins of the gear wheel of the head (CS 2269-xx) engage in the recesses of the expansion ring on the implant.

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Now place the head (CS 2269-xx) and the assembly assistant (CS 2256-02) on the implant and ensure correct positioning of the screw and the gear wheel in the recesses provided on the implant.

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**Important:** Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to: UH 1100 „Processing manual implants and instruments“. For individual instruments which are correspondingly marked (Δ), we refer to the „Assembly and disassembly instructions with special cleaning instructions“. These can be downloaded any time free of charge at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)

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#### Mounting the head in the implant

Now hold the implant in one hand and the head (CS 2269-xx) and the assembly assistant (CS 2256-02) in the other hand and ensure that the correct positioning is maintained. Rotate the assembly assistant (CS 2256-02) clockwise to firmly secure the head to the implant.

Remove the assembly assistant (CS 2256-02) from the posterior end of the head (CS 2269-xx). Test the expansion function by rotating the external hexagon connection on the posterior working end of the head (CS 2269-xx).

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#### Assembly of the distractor

Insert the inner shaft, short, 214 mm (CS 2256-21) into the outer shaft, short, 128 mm (CS 2256-20).

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Attach the handle (CS 2256-01) securely to the inner shaft, short, 214 mm (CS 2256-21).

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**Important:** Please ensure that you are using and observing the most current instructions for use and surgical technique of this system at all times. For the cleaning, disinfection and sterilization of unsterilized implants and instruments, we refer to: UH 1100 „Processing manual implants and instruments“. For individual instruments which are correspondingly marked (△), we refer to the „Assembly and disassembly instructions with special cleaning instructions“. These can be downloaded any time free of charge at: [www.ifu.ulrichmedical.com](http://www.ifu.ulrichmedical.com)

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#### Assembly of the distractor

Insert the implant locking rod, short, 276 mm (CS 2256-22) through the handle (CS 2256-01) into the instrument. Ensure that the implant locking rod, short, 276 mm (CS 2256-22) remains able to move freely.

Take the head (CS 2269-xx) with the implant attached to it and insert the posterior end of the head (CS 2269-xx) fully from the front into the outer shaft, short, 128 mm (CS 2256-20).

Secure the head (CS 2269-xx) on the instrument by rotating the knurled part of the outer shaft, short, 128 mm (CS 2256-20) clockwise until it is positioned securely over the head.

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#### The fully assembled distractor

Now test the expansion function once again by rotating the blue handle clockwise.

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#### Inserting the implant using the distractor

The previously attached implant is inserted in the surgical site with the aid of the distractor (CS 2256). Rotate the blue handle clockwise to expand the implant. If you have reached the required height, rotate the implant locking rod, short, 276 mm (CS 2256-22) counterclockwise until the implant is released from the instrument. Remove the distractor from the surgical site.

If the implant in question is an ADD™ implant, continue with the "Implant fixation ADD™" surgical step from the system-related surgical technique on page 14.

If the implant in question is an ADDplus™ implant, continue with the "Assembly of depth stop" surgical step from the system-related surgical technique on page 15.

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#### Disposal of used medical devices

For disposal of used medical devices, please observe the country-specific regulation for disposal of clinic waste.

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